



INSTRUCTIONS FOR USE SOMNA® COLLAR



The Somna® Collar is a daytime product that helps to reduce tension in the body and shoulders.

USE

The collar is ergonomically designed to provide even, stable pressure. By lying on the chest and going down between the shoulder blades, it applies pressure to the muscles and gives the user a pleasant feeling of being embraced.

The collar can be used both when sitting still and when moving. The collar helps to ensure better posture, which promotes circulation and balance in the body.



WASHING INSTRUCTIONS

Always use the accompanying laundry bag.



Maximum 60°C - gentle wash cycle.



Do not iron



Do not bleach



Do not dry clean



Do not tumble dry

PRODUCT INFORMATION



CE approved to the Medical Devices Directive 93/42/EEC.



The Somna® Collar is a Swedish registered design.

WEIGHTS

Junior	1 kg
XS	1,5 kg
S-XL	2 kg kg

MATERIALS

Textile: 55% Polyester, 45% Cotton

Filling: 100% Polyester

ACCESSORIES

A protective cover in polyester cotton is available to purchase as an optional extra. The cover can be washed at 60°C maximum.

NOTE

- The collar must not be heated.
- Observe mispositioning. Do not place on the throat when lying down due to choking hazard.