



DAY



2-1 FUNCTION

## INSTRUCTIONS FOR USE SOMNA® COMFORTER

Settle down with the Somna® Comforter. The chains inside the lengthwise channels shape the comforter tightly to the body's contours, giving the user a pleasant feeling of being embraced.



### USE

The Somna® Comforter has chains stitched into lengthwise channels. The chains are attached at both ends and cannot be removed. The steel chains are painted and therefore completely silent.

The comforter has a two-in-one function: one chain side and one padded side. Which side is against the body is down to personal choice. Having the chains against the body provides greater tactile stimulation. Try out both sides to see what's best for you.

The comforter is so supple that you can also fold it if you require more weight on a certain area of the body. The comforter is designed to be as cool as possible.

## WASHING INSTRUCTIONS

Always use the accompanying laundry bag.



Maximum 60°C – gentle wash cycle.



No liquids stronger than PCE when dry cleaning



Do not bleach



Reconditioning is possible at 70°C



Do not tumble dry



Do not iron



## PRODUCT INFORMATION



The comforter is made of material which fulfils SS-EN ISO 12952-1 standards (Ignitability of bedding items – Ignition source: smouldering cigarette).

**Textile:** 75% Polyester, 17% Cotton, 8% Polypropylen

**Filling:** 100% Steel chains



The Somna® Comforter is CE approved to the Medical Devices Directive 93/42/EEC.



The Somna® Comforter is a Swedish patented product.

## WEIGHTS & DIMENSIONS

Somna® Comforter 4 kg:	70 x 125 cm
Somna® Comforter 5 kg:	70 x 200 cm
Somna® Comforter 7 kg:	70 x 200 cm

### NOTE

- If the user has reduced lung capacity, seek medical advice before using a weighted comforter.
- Consult a doctor if the user has any surgically inserted devices that may be affected by the steel in the chains.
- Beware the risk of tripping if the comforter is left on the floor.
- Take care if the user has low muscle capacity.